



\*not provided by MMD

# ORACOL INSTRUCTIONS FOR USE

**Products Covered:**

This IFU covers the Oracol (S10) saliva collection device.

**Intended Use:**

Intended for use in the collection of saliva samples for use in vitro diagnostic tests.

**Intended User:**

The devices are intended to be used by medical professionals and members of the public.

**Patient Population:**

Indicated for use in adults and paediatrics.

**Packaging & Sterilisation:**

The devices are supplied sterile, for single use only. Each device is sterilised via gamma irradiation in accordance with ISO 11137.

**Precautions/contraindications:**

- Not to be used if the patient has a bleeding gum.
- The device is single use only. Reuse may cause infection.
- Do not use the device if the device is damaged.

**Storage:**

- To be stored under normal ambient conditions.
- The shelf-life of the product is stated on the label.

**Disposal:**

Dispose of the device in accordance with local clinical disposal instructions.

**Instructions:**

Refer overleaf for the instructions

**Name and address of manufacturer:**

Malvern Medical Developments Ltd.

Unit 10, Northbrook Close

Barbourne, Worcester

WR3 8BP, United Kingdom

**Tel:** +44 (0)1905 731343

**Email:** sales@malmed.co.uk

**Name and address of EC Rep:**

Advena Ltd.

Tower Business Centre, 2nd Flr.,

Tower Street, Swatar

BKR 4013, Malta.